



NITE Owl

CPAP BATTERY

Rest Assured with NiteOwl
*Your companion for anytime
anywhere sleep therapy!*

Camping

Maintain your sleep therapy when camping or enjoying other travel adventures.



Option to connect two NiteOwl batteries together for longer run times.

Power Outages



Battery backup system for uninterrupted sleep therapy during a power outage.

*Powers your CPAP machine for 1-3 nights with the heated humidifier turned off. *Run times depend on many factors including device type, pressure setting, mask fit, etc.*

Air Travel

FAA-approved for use on board an aircraft making it great for international & red-eye flights.



Small and lightweight so it's easy to keep with your CPAP machine and supplies.

3.0 USB PORT



Quick-charge 3.0 USB port charges your go-to electronic devices.

Charge the NiteOwl with your CPAP power supply - no need for extra cords.



Dimensions: 7.5" L x 4.8" W x 0.8" H
Weight: 1.7 lbs.

SPECIFICATIONS

Cell Type: Rechargeable Lithium-ion

Casing: Aluminum Alloy

Power Gauge: Digital Display (shows charge level as a %)

Capacity: 99.9 Wh

Output Voltage: 12V-24 DC (depending on DC output cable)

Output Current: 8A (maximum)

DC Input Voltage Range: 10V-28V

QC 3.0 USB Output Voltage: 5V | 2.4A (maximum)

Charge Input: 10V-28V

Optimal Operating Conditions: 23°F to 104°F

**capacity is reduced by 30% when operating between 23°F to 32°F*

Optimal Charging Conditions: 32°F to 104°F

Storage Conditions: -4° to 104° (optimal range 60°F-80°F)

Charge Time: 2-3 hours @ 90W

**charge times are per battery using a 90W rated AC power supply*

Life Span: 500 cycles (100%-0% then back to 100%)

**after 500 cycles the battery shall recover at least 70% of initial charge*

Warranty: 1 year limited



866-718-0557



sleepniteowl@gmail.com



niteowlcpapbattery.com



NITE Owl
CPAP BATTERY



Does NiteOwl work with all PAP devices? NiteOwl works with the most common makes and models of PAP devices. Some targeted ventilation PAP devices, as well as obsolete makes and models, may not work or may require more power than NiteOwl is designed to provide.

How long will NiteOwl power my PAP device on a full charge? Typically you will get 1-3 nights of use when using a single NiteOwl battery; however, many factors can impact run times. The type of device, pressure settings, altitude, and more can greatly affect how long you can power your PAP device. Using a heated humidifier or tubing, for example, will most often reduce the run time by over 65% and, in some cases, can prevent NiteOwl from powering the device altogether.

Is NiteOwl covered by my insurance? Some insurance companies will cover the NiteOwl battery under HCPCS code A4601. However, because NiteOwl is not Medicare coded and is not considered a medically necessary device, it is unlikely to be covered by many insurance companies.

Can I use NiteOwl as a battery backup system in case of a power outage? Yes. NiteOwl can be set up as a battery backup system and will automatically turn on if power is lost, which allows your sleep therapy to continue uninterrupted.

Can I take NiteOwl on an airplane? Yes. NiteOwl meets all current FAA requirements for lithium-ion batteries that are allowed in carry-on baggage (e.g., 100Wh or less per battery).

Can I use NiteOwl to power other devices? Yes. The quick-charge 3.0 USB port featured on the NiteOwl is perfect for your go-to devices like cellphones and tablets.

How long does it take to charge NiteOwl? Charge times range from 2-3 hours if using a 90W power supply but will vary depending on the wattage of the AC power supply being used.

Can I charge NiteOwl in a vehicle or with solar power? Yes. NiteOwl sells an Auto Charge Cable for quick and easy charging while you are on the road. There are many other portable charging and solar solutions available on the market. Check the NiteOwl input specifications for compatibility.

Does NiteOwl meet safety regulations and shipping requirements? Yes. The NiteOwl battery is UN38.3 certified, meets UL2056 testing standards, and all current safety and shipping regulations.

Where can I buy a NiteOwl and accessories? Ask about the NiteOwl CPAP battery at your local sleep therapy equipment provider or contact us and we'll help you find a retailer in your area.



866-718-0557



sleepniteowl@gmail.com



niteowlcpapbattery.com